




permobil



THE ROAD TO INDEPENDENCE

Permolock is a device for securing power wheelchairs in place inside a vehicle, which allows users to actually drive while sitting in their wheelchairs. Permlock is designed specifically for use with Permobil's C400 and C500 power wheelchair bases and consists of two elements: a locking device mounted to the wheelchair chassis and a locking plate mounted to the vehicle. These features ensure the wheelchair does not break free during a collision or because of some other sudden movement. Both elements are easily controlled with the simple touch of a button.


PERMOLOCK

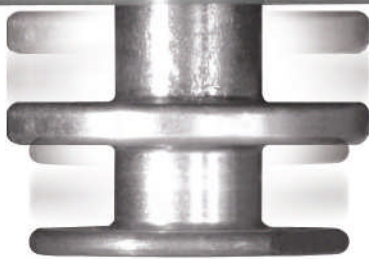


RETRACTABLE LOCKING PIN
Activated by Control Button and drops into place to secure the wheelchair to the base. Retracts when not in use.



CONTROL BUTTON
Located on the armrest and used to drop the locking pin into place, so it can properly dock to the locking plate.

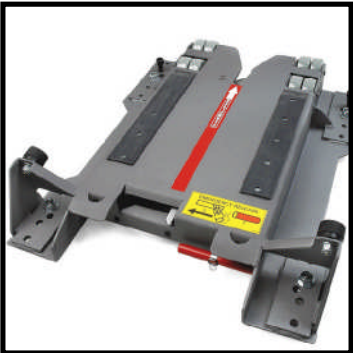


LOCKING PLATE
Easy-to-install docking station that holds the chair firmly and securely in position.

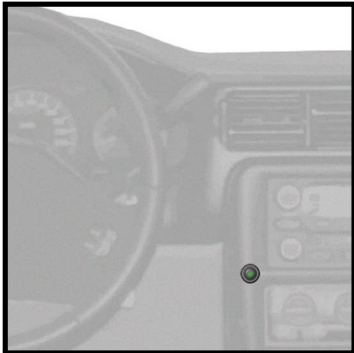

VEHICLE CONTROL BUTTON
Located on the dash, it uses three lights—red, amber and green—to indicate if the chair is locked into place. Simply press and hold to unlock the wheelchair.



EMERGENCY RELEASE
Locking mechanism can be opened manually with the emergency handle on the front edge of the locking plate.

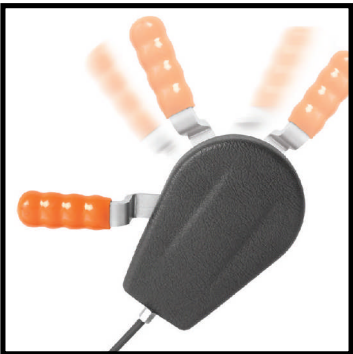


MANUAL RELEASE
Wheelchair can be released manually in the event of vehicle power failure.

ACCESSORIES



CHAIR BASE
Permits use of original car seat in Permlock.

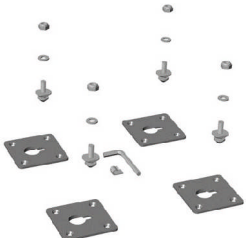

QUICK ANCHORING SET
Used to anchor the Permlock in a vehicle and allows easy transfer of the lock between different positions in the vehicle.



INSTALLATION SPACER
Used for installing Permlock on a sloping surface.



THE POWER OF MOBILITY

Permobil Inc.
6961 Eastgate Boulevard
Lebanon, Tennessee 37090
800.736.0925
www.permobil.com

* Please consult owner's manual before using Permlock.

Permlock must be installed by a certified NMEDA representative.