

Explorer Mini

How to clean and disinfect

If the Explorer Mini is used by more than one child, cleaning & disinfecting before and after use is recommended. For more information, please refer to cleaning methods as outlined in the owner's manual.

NOTE: Keep power off during cleaning and disinfecting. Do not allow control panel or electronic components to be soaked with water or other fluids. Fluids entering electronic components can permanently damage your product.

Cleaning the frame and joystick

1. Hand wash using: mild liquid hand soap, hand dishwashing liquid, laundry detergent, or a multipurpose detergent.
2. Gently scrub all surfaces, using a soft brush, sponge or cloth.
3. Use a cloth dampened with clean water to rinse and remove cleaning agent.
4. Dry thoroughly with a clean soft cloth.

Disinfecting the frame and joystick

1. Use a soft cloth dampened with bleach solution (1 part household liquid bleach per 9 parts water) or EPA registered disinfectant.
2. Allow the bleach solution or disinfectant to sit for 10 minutes.
3. Use a cloth dampened with clean water to rinse and remove cleaning agent.
4. Dry thoroughly with a clean soft cloth.

Cleaning the backrest upholstery cover

1. Remove the upholstery from device and take out the foam from the cover.
2. Machine wash cover only in cold water 86°F (30°C) with mild detergent on the gentle cycle, or wipe clean with neutral detergent and warm water.
3. Tumble dry low.

Disinfecting the backrest upholstery cover

1. Hand wash in cold water 86°F (30°C) using 1 part household liquid bleach per 9 parts water.
2. Keep the product wet with the bleach solution for 10 minutes, then rinse thoroughly. Or machine wash in hot water 140°F (60°C).
3. Tumble dry low.

Cleaning the back upholstery foam

1. Wipe lightly with a damp cloth. DO NOT use soap and DO NOT submerge in water.
2. Wipe off with a clean cloth.
3. Completely air dry before reassembly.
4. Replace soiled or contaminated foam with authorized Permobil replacement parts only. DO NOT use soiled or contaminated foam with multiple users.

NOTE: Disinfectants are not effective on porous surfaces such as foam.

Please follow product label instructions and safety guidelines for all cleaning and disinfectant products and use personal protective equipment as recommended.